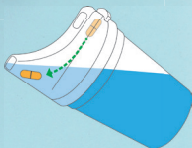
Hydas

Schnabelbecher

«PILLENSCHLUCKER»

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Lid, Pill shoot, cover, user manual

2 | Intended purpose

- Ideal for children and people with disabilities
- Simplifies the swallowing of tablets and capsules. The tablet is „washed“ down the throat and easily is swallowed
- No sticking to tongue or on the palate and no gag reflex
- Even swallow pills while lying down
- Ensures that pills are taken with sufficient liquid

3 | Notes on safe use of the product

- Not dedicated to be used unsupervised by children, individuals with sensory deprivation of any kind or mentally challenged individuals. Any users falling into these categories should be assisted by a responsible adult.
- **Attention:** People with swallowing difficulties (Dysphagia) should consult their physician before using product.
- When the pill shoot is damp, the pill or tablet could stick, therefore keep tablet shoot dry.

4 | Application of the product

Clean the device before first use to remove dirt and debris.

- Slide the pill shoot (2) into the opening (2a). Be sure to insert all the way to the hilt.
- Fill the cup with sufficient liquid. Maximally, 250 ml fill to mark on cup.
- Place the lid (1) on the cup (3) and press until a light click is heard. Only then may one be assured the lid not to leak.
- Lay the medicine (pill, tablet, capsule) into the receptacle (1a).
- Place the opening to the lips and swallow the medicine with the accompanying liquid.

Attention: After taking a pill, ensure that it has been cleared from the shoot. Remove the lid and the shoot from the cup.

Tip: In order to easily remove the lid from the cup, place thumb under the inserted pill shoot and push up.

5 | Cleaning

- By hand or in the dishwasher
- Possible to clean all components separately

Materials: Polypropylene free of all softening agents, food safe, dishwasher safe.

6 | Warranty

The guarantee lasts for 24 months from the date of purchase for manufacturers' error in material and quality. If it is used for another purpose or operated incorrectly, no liability for possible damages can be accepted. This guarantee is only valid if the instructions in the "instructions for use" have been followed. Warranty is void if product is misused or repairs attempted by unqualified persons.

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Hydas GmbH & Co. KG
c/o atrikom fulfillment GmbH • Haagweg 12