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The SoftAirSeat Pro is the further development of our established SoftAirSeat Hybrid. The SoftAirSeat Pro seat cushion has an integrated air pump to inflate the seat cushion and an integrated deflate valve to release the air. One of the great advantages is that the amount of air in the seat cushion can be adjusted while you remain seated on it.

The 20 individual air cushions, which are arranged in a specific design to relieve the sitting bones and coccyx, create a very comfortable sitting experience. After unpacking the seat cushion, simply press "INFLATE" with the flat of your hand until there is enough air in the seat cushion.

Experience has shown that approx. 50-60% of the maximum air volume creates an optimum seat comfort.

To release the air, please remain seated on the seat cushion and press the "DEFLATE" valve. This will release approx. 95% of the air from the seat cushion. After folding, press the deflate valve again to press the remaining air out of the seat cushion. It can then be packed back into the transport bag.

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SoftAirSeat **Pro**

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A product of **TECSEAT®**
innovation for seating

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TecSeat GmbH